# PROVOX

## Skin Barrier

# Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Ingredients/ Inhaltsstoffe/ Ingrediënten/ Contient/ Ingredienti/ Ingredientes/  
Ingredientes/ Innehållsämnen/ Indholdsstoffer/ Innhold/ Ainesosat/ Innihaldsefni/  
Koostis/ Sudėtis/ Složení/ Összetevők/ Zloženie/ Sestavine/ Składniki/ Ingrediente/  
Sastojci/ Sastojci/ Συστατικά/ Составки/ Ингредиенты/ İçerik/ רכיבים/ المكونات/  
Bahan-bahan/ 成分/ 성분/ מרכיבים/ المكونات :

Hexamethyldisiloxane, Isopropyl  
myristate, Trimethylsiloxysilicate

## Contents

EN - ENGLISH.....	4
DE - DEUTSCH.....	5
NL - NEDERLANDS.....	6
FR - FRANÇAIS.....	7
IT - ITALIANO.....	8
ES - ESPAÑOL.....	9
PT - PORTUGUÊS.....	10
SV - SVENSKA.....	12
DA - DANSK.....	13
NO - NORSK.....	14
FI - SUOMI.....	15
IS - ÍSLENSKA.....	16
ET - EESTI.....	17
LT - LIETUVIŲ KALBA.....	18
CS - ČESKY.....	19
HU - MAGYAR.....	20
SK - SLOVENČINA.....	21
SL - SLOVENŠČINA.....	22
PL - POLSKI.....	23
RO - ROMÂNĂ.....	24
HR - HRVATSKI.....	25
SR - SRPSKI.....	26
EL - ΕΛΛΗΝΙΚΑ.....	27
RU - РУССКИЙ.....	28
BG - БЪЛГАРСКИ.....	29
TR - TÜRKÇE.....	30
HY - ՀԱՅԵՐԵՆ.....	32
MS - BAHASA MELAYU.....	33
JA - 日本語.....	35
KO - 한국어.....	36
ZHTW - 繁體中文.....	38
HE - עברית.....	39
AR - العربية.....	40

## Instructions for use

### Intended Use:

Provox Skin Barrier is a single use wipe for laryngectomized patients that forms a barrier between Provox Adhesive and the skin.

### CONTRAINDICATIONS:

Hypersensitive or breached skin or known allergies to ingredients.

### Application:

Clean the skin around the stoma and let dry. Apply the Provox Skin Barrier to the skin around the stoma that will be covered by the adhesive. Allow the barrier to dry. Apply a Provox Adhesive.

### PRECAUTIONS

- Avoid deep inhalation of solvent during application and be careful that liquid does not drip into stoma.
- In case you experience signs of irritation or redness, discontinue use of product. If the irritation persists, consult your physician.

### WARNINGS

- For external use only;
- Keep out of reach of children;
- Avoid contact with eyes. In the case of accidental contact, flush eyes well with water;
- Do not apply directly to open wounds and mucous membranes;
- Single patient use only. Re-use may cause cross contamination.
- Do not cut the tissue in pieces, small pieces may be aspirated

### Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

### Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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Atos and the Atos logo are trademarks of Coloplast A/S.
© 2026 Coloplast A/S. All rights reserved.
Atos Medical AB